

Car loop instructions – CLS-1

Contents: Univox CLS-1 Amplifier, 12V DC car power adapter, chairpad, lapel microphone



Amplifier



DC Car Adapter



Chairpad



Microphone

Installation:

1. Insert stripped ends of the chairpad into the red and black receptors on the rear of the amplifier
2. Connect the microphone to plug located at the rear of the amplifier
3. Connect the car power adapter to a car outlet and to the power plug at the rear of the amplifier
4. Place the chairpad on the car seat
5. Switch your hearing aid to t-coil (T/MT)
6. Test the microphone and adjust the volume and tuning dials on the front of the amplifier to your satisfaction.